


# Surgical Clipper Setup Guide\*

## Surgical Clipper Blade Options

Description	Cat. No.	Qty.
General Purpose – for body hair	4406	50/cs
Specialty – for scalp and other thick, coarse hair	4412A	20/cs

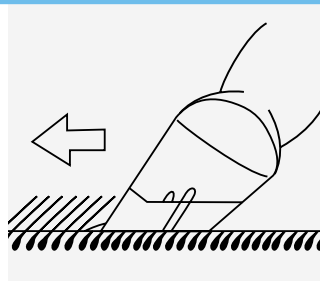

## Clipping

With blade at 60° angle to skin, glide blade against surface of skin. For best results, clip against lie of hair. For difficult areas, stretch skin taut.


## Attaching

Place blade on top of clipper head. Slide blade back until indentations on clipper's side are aligned.



## Removing

With blade facing downward, use thumb to push blade forward. Allow used blade to fall into appropriate receptacle.

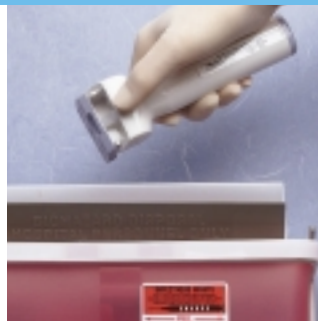

## Cleaning

Rinse clipper under warm or cold running water or submerge in water. Dry surface with clean towel.




## Disinfecting

Wipe with surface disinfectant or soak in antimicrobial agent. Then, rinse with water and towel dry.



## Charging

Switch clipper to off position. Place clipper handle in charging adapter. (Full charge requires 12 hours.)

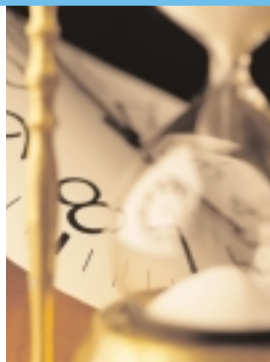

## Storing

Store clipper in upright position utilizing charging adapter 4414.



\* Refer to clipper packaging insert for more detailed instructions.  
Note: To avoid cross contamination, use a new blade for each patient.