

Medi-vac Flex Advantage™

Suction Waste Retention System

Assembly



1 Insert the liner correctly into the canister and to assure a good seal, press firmly downwards around the rim of the lid, whilst supporting the neck of the canister. You should hear and feel it click into place. This creates the vacuum around the outside of the liner.



2 Check all caps are covering unused ports for a proper seal. Insert the red vacuum tube to the port labelled Vacuum, (it will not fit in any of the other ports). This creates the vacuum inside the liner.



3 NOTE: When the liner is seated correctly in the canister, it creates equal vacuum inside and outside the liner, which will obtain maximum suction potential. You can now test that there is adequate suction at the patient port, by switching the white bar to ON.



4 Connect the patient suction tubing to the right-angled port or directly onto the patient port. The right-angled port is a removable component, which is used to prevent the suction tubing kinking, if the canister is fixed at a height well above the patient.



5 To discard the used liner, first add the gel (if this is your disposal method of choice), and then ensure all caps are firmly on the correct ports. The last port should be the vacuum port, before finally switching the vacuum source off.

IMPORTANT the vacuum source should remain ON until all ports are in place. Once the vacuum is off then the liner can be safely removed by placing one hand on the neck of the canister and raising the edge of the liner.



6 Please note that the liners have their size, a lot number and reference code on the lid to assist you in identifying the liners. There are three sizes

1000cc 65651910
1500cc 65651920
3000cc 65651930

Liners should only be placed in the correct canister size, to use a different size liner to that of the canister could affect the liners performance.

Ordering Information

Flex Liners	Code No.	Pack Size
1000cc Liner	65651910	50
1500cc Liner	65651920	50
3000cc Liner	65651930	50
Disposal Boxes	FKD0032FL	10
Gel-Vac	RE2015	2 x 100 Sachets
Disposal Bin 50L	FKD0033	Each


CardinalHealth
Medical Products & Services
Frankland Road, Blagrove
Swindon, Wiltshire SN5 8RU
Tel: Customer Services 0870 6011011
Fax: 0870 6011937