




APPLICATION

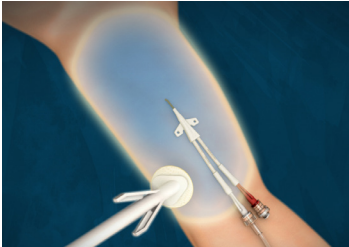

Thoroughly prep **beyond** entire dressing site according to facility protocol.

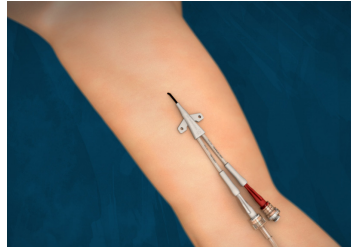


Allow Chloraprep® to **dry completely** per manufacturer's instructions.

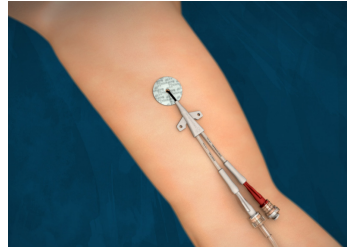


If required apply BIOPATCH® Protective Disk over insertion site blue and printed side up.




Remove larger liner without touching adhesive.

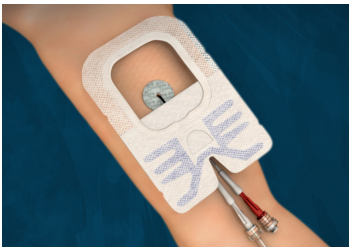

Center the insertion site in the large window.

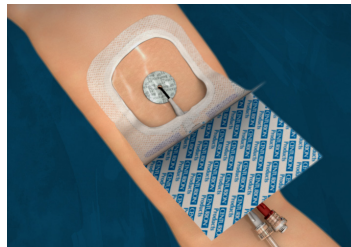


Smooth down to adhere, then remove remaining liner.

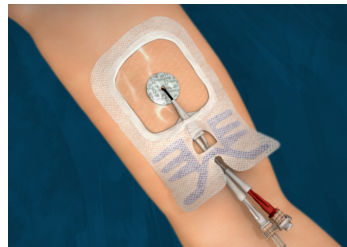

Guide the tubing through the notch.




Overlap the edges to form a tight seal and smooth down to adhere.

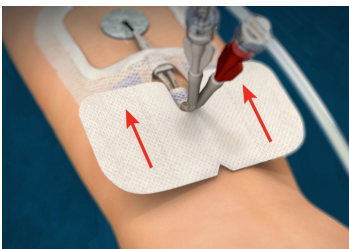


Slip closure piece **under** tubing and **over** the edge of the dressing.



Remove the liner on one side and smooth down to adhere.

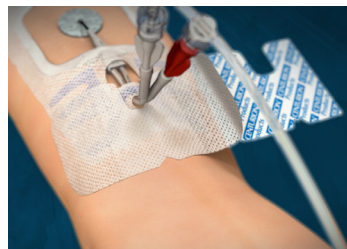

Remove the liner on remaining side and smooth down to adhere.

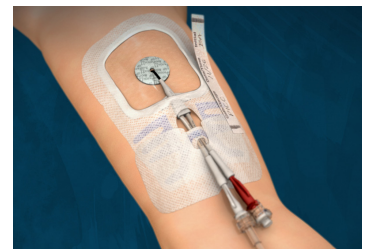


Record information on label and position on the edge of dressing.

REMOVAL



Optional: A tape strip may be used to secure catheter tubing during removal.

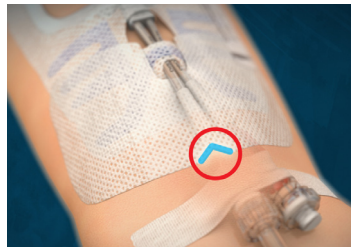

Locate the v-notch on the outer edge of the closure piece and pull apart to break center perforations.



Holding the catheter in place, peel back the closure piece and dressing together as one piece.



Peel back slowly, keeping dressing close to skin and following catheter toward insertion site.

Customer Services and Distribution Contact Details

Ireland
ROI Tel: +353 1 404 8383
NI Tel: 0845 850 8600
Email: info@iskushealth.com

United Kingdom
Tel: +44 (0) 1256365458
Email: uksales@iskushealth.com