

WINGGUARD®
CATHETER SECUREMENT DEVICE

Patient Friendly
Nurse Friendly
Dressing Friendly



“improved adhesive products and securement devices... are essential to provide increased catheter stability.

Such products... may reduce catheter dislodgement and the necessity of reinsertion with its associated needle stick risk.”

- From the OSHA *FactSheet: Securing Medical Catheters*

BENEFITS FOR CAREGIVERS, HOSPITALS AND PATIENTS

- Silicone top layer allows for clean removal of dressings
- Extremely easy to apply and remove
- Universal size secures any size or type of PICC catheter
- Low-profile design minimizes bumping and jostling to the site
- Pliable construction conforms comfortably to body contours
- Patient-friendly adhesive is gentle, yet stays in place for long wear times
- Available in two latex-free materials

APPLICATION INSTRUCTIONS

Thoroughly clean and prepare site. Prep well beyond entire site according to facility protocol to ensure adherence. Allow to dry completely (for at least 30 seconds). Note: If the prep doesn't dry completely, the integrity and security may be compromised. A skin protectant may be used according to preference or protocol. Allow to dry completely.

Slide WingGuard under catheter hub and center hub wings over WingGuard.

Holding catheter in place, pull tab out from center to create an opening in slit. Insert first catheter wing into slit.

Repeat on other side.

Remove liner one side at a time and adhere WingGuard to skin.

Apply preferred dressing over the site as usual.


REMOVAL INSTRUCTIONS (not shown)

Remove dressing.

Optional: Apply sterile tape strip to catheter tubing to help stabilize catheter prior to removing WingGuard.

While stabilizing the catheter, pull WingGuard tab out from center to create an opening in slit to release catheter hub wings from device.

Peel WingGuard slowly off skin, away from exit site.

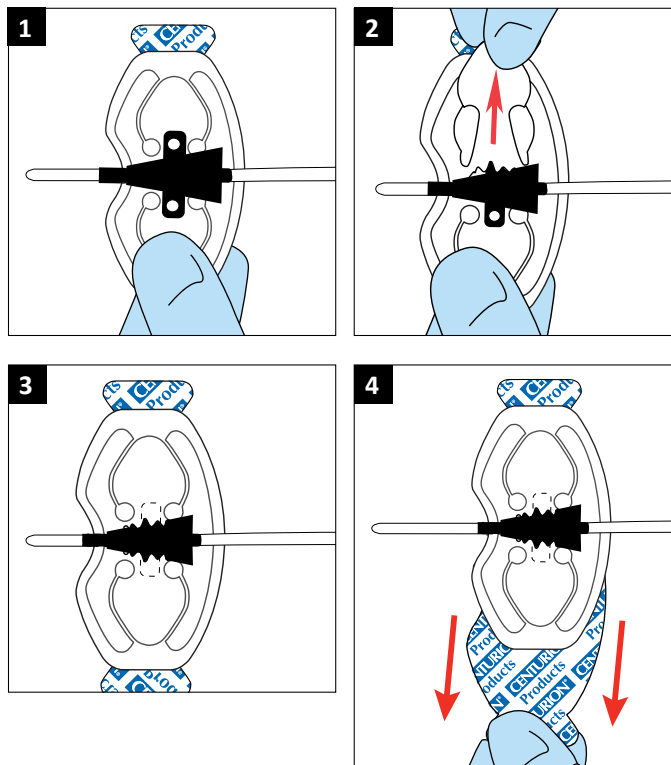

To properly dress the site, a dressing should cover the catheter exit site.

WingGuard® is a registered trademark of Centurion Medical Products Corporation.



Customer Services and Distribution Contact Details

United Kingdom

Tel: +44 (0) 1256365458

Email: uksales@iskushealth.com

Ireland

ROI Tel: +353 1 404 8383

NI Tel: 0845 850 8600

Email: info@iskushealth.com

www.iskushealth.com