

# ChloroPrep

## Venous access and maintenance range

2% Chlorhexidine Gluconate w/v and 70% Isopropyl Alcohol v/v

Licensed, single use, sterile, ANTT™, latex free, skin antiseptic system

### Pinch

Holding the barrel of the applicator, pinch the levers together, being careful not to touch the sponge.

Pinch the lever to release the solution. You will hear a 'pop' as the ampoule breaks.



### Apply

Starting at the incision site, gently press the applicator against the skin until the solution soaks the sponge. Apply using repeated up and down, back and forth strokes for at least 30 seconds, before working outwards to the periphery.



### Dry

Leave the area to air **dry completely** before applying sterile drapes. Do not blot or wipe away. Discard the applicator after a single use.

#### Important safety point

Do not drape or use ignition source until the solution has completely dried.



### 3ml Tint

**Coverage area:** 15 cm x 15 cm

**For procedures such as:**

- Central venous catheter insertions
- Insertion site maintenance
- Midline catheter insertions
- Minor procedures

Product code: 270415



### 10.5ml Tint

**Coverage area:** 25 cm x 30 cm

**For procedures such as:**

- Surgical site antiseptic
- Tunelled central venous catheter insertions
- Interventional radiology
- Implantable device placements

Product code: 270715



**ChloroPrep®**  
Patient Preoperative Skin Preparation  
2% Chlorhexidine Gluconate (CHG) w/v  
& 70% Isopropyl Alcohol (IPA) v/v

**For customer services and all other enquiries:**

Please telephone: +353 1 404 8383

Email: [info@iskushealth.com](mailto:info@iskushealth.com)

or visit: [www.chloroprep.co.uk](http://www.chloroprep.co.uk)

Distributed by



**ISKUSHEALTH**

### Prescribing Information

Chloraprep with Tint (PA-1435/001/001) 2% chlorhexidine gluconate w/v / 70% isopropyl alcohol v/v cutaneous solution. **Indication:** Disinfection of skin prior to invasive medical procedures. **Dosage & administration:** Chloraprep with Tint – 3ml, 10.5ml, 26ml. Volume dependent on invasive procedure being undertaken. Applicator squeezed to break ampoule and release antiseptic solution onto sponge. Solution applied by gently pressing sponge against skin and moving back and forth for 30 seconds. The area covered should be allowed to air dry. **Side effects, precautions & contra-indications:** Very rarely allergic or skin reactions reported with chlorhexidine, isopropyl alcohol and Sunset Yellow. Contra-indicated for patients with known

hypersensitivity to these constituents. For external use only on intact skin. Avoid contact with eyes, mucous membranes, middle ear and neural tissue. Should not be used in children under 2 months of age. Solution is flammable. Do not use with ignition sources until dry, do not allow to pool, and remove soaked materials before use. Over-vigorous use on fragile or sensitive skin or repeated use may lead to local skin reactions. At the first sign of local skin reaction, application should be stopped. **Legal category:** P. **Marketing Authorisation Holder:** CareFusion UK 244 Ltd, 43 London Road, Reigate, Surrey RH2 9PW, UK. **Date of preparation:** July 2010.  
CHL135 August 2010