


APPLICATION

Prep the skin with ChloraPrep® according to the protocol. Allow ChloraPrep® to air dry completely



Peel back larger liner halfway, align catheter with lower edge of window. Press to adhere to catheter and skin. Completely remove larger liner. Remove small liner, adhere dressing to skin. (Figure A).

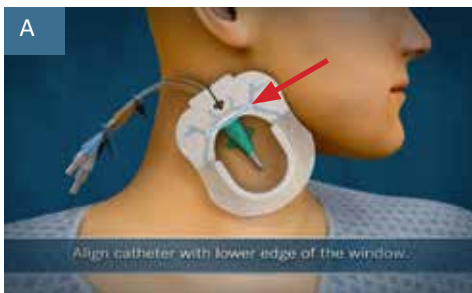

Guide tubing out through center notch and overlap the inner tab edges to form a seal (Figure B).



Without removing the liner, slip the closure piece **under** tubing and **over** edge of dressing as far as possible (Figure C). Remove the liner one side at a time and smooth down to adhere.

REMOVAL



Locate v-notch on outer edge of closure piece and pull apart in opposite directions to break center perforation (Figure D).



Pull apart completely (Figure E).

Optional: A tape strip may be used to secure catheter tubing during removal (Figure E).



Holding the catheter in place, gently peel back closure piece and dressing together as one. Peel back slowly, keeping dressing close to the skin, following catheter toward insertion site (Figure F).

Customer Services and Distribution Contact Details

Ireland
ROI Tel: +353 1 404 8383
NI Tel: 0845 850 8600
Email: info@iskushealth.com

United Kingdom
Tel: +44 (0) 1256365458
Email: uksales@iskushealth.com