

APPLICATION

ChloraPrep® well beyond entire dressing site according to facility protocol to ensure dressing adherence. **Allow ChloraPrep® to air dry completely** per manufacturer's instructions.

Center the insertion site in window, with the connecting hub within or under the blue SHIELD technology (**Figure A**). Adhere dressing to skin.

Remove remaining liner. Guide the tubing out through notch. Pinch the edges of the notch together to form a seal (**Figure B1**).

Exit site edges should be touching or overlapping (**Figure B**). From center outward, firmly smooth down entire dressing to ensure all edges are adhered well to the patients skin.



Without removing the liner, slip the closure piece **under** tubing and **over** edge of dressing (**Figure C**). Remove the liner one side at a time and smooth down to adhere.

Record information on label, remove from liner and position label on the edge of the dressing.

REMOVAL

Optional: A tape strip may be used to secure catheter tubing during removal.

Locate v-notch on outer edge of closure piece (**Figure D**) and pull apart in opposite directions to break center perforations (**Figure E**).

Holding the catheter in place, gently peel back closure piece and dressing together as one piece (**Figure F**). Peel back slowly, keeping dressing close to the skin, following catheter toward insertion site.

Optional: Use an alcohol prep pad **underneath** dressing while peeling back to help deactivate the adhesive.



Customer Services and Distribution Contact Details

Ireland
ROI Tel: +353 1 404 8383
NI Tel: 0845 850 8600
Email: info@iskushealth.com

United Kingdom
Tel: +44 (0) 1256365458
Email: uksales@iskushealth.com