


# FlexAgrip™ Catheter Securement Device


CODE CSD0119

Application


Prepare the site according to your facility's protocol. Allow ALL preps to dry completely. Remove sterile dressing from package.

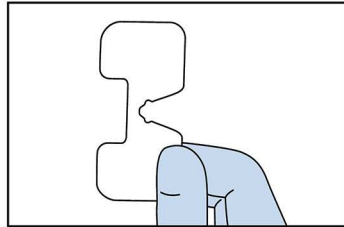


Obtain FlexAgrip

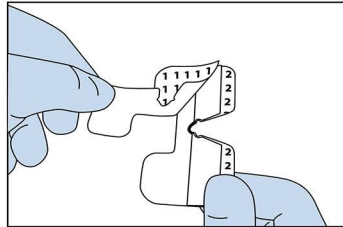


Remove liner on rectangular side

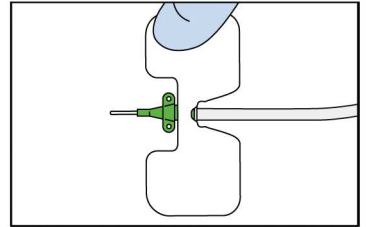


Place FlexAgrip behind catheter wings

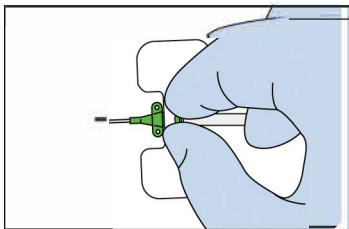


Flex or pinch around hub

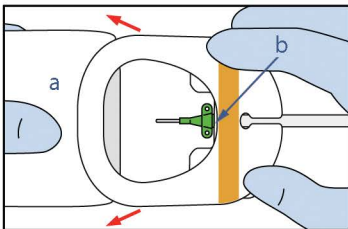


Obtain dressing & remove liner (a). Place lower edge window (b) behind wings.

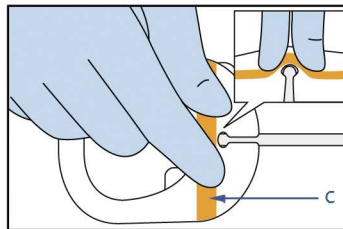


Remove smaller liner and pinch the center of the shape memory material (c) to contour around the catheter.

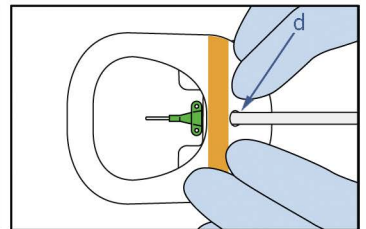


Guide tubing out through notch (d) and adhere remaining portion with firm pressure.

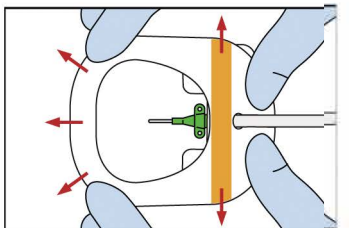


Apply firm pressure to the entire dressing, starting from the center to outer edge, to ensure optimal adhesion.

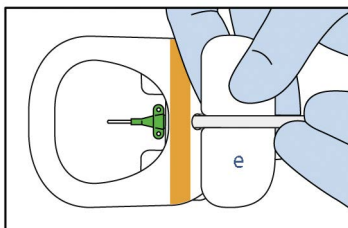

Position Secure Strip (e) under tubing and over dressing edge, then remove the liner and smooth down to adhere. Apply firm pressure to the Secure Strip.

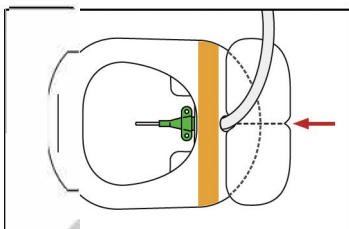


Locate v-notch on outer edge of the Secure Strip and pull apart in opposite directions to break center perforations.

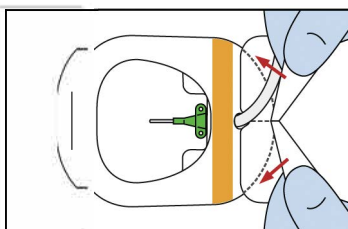

Holding the catheter in place, gently peel back Secure Strip and dressing together as one. Peel back slowly, keeping dressing close to the skin, follow catheter toward insertion site.

Removal

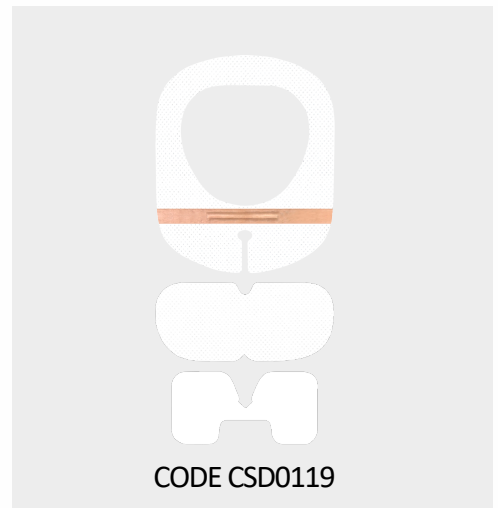

**ISKUS HEALTH**  
IMPROVING PATIENT OUTCOMES

Tel: +44 (0) 208 869 6509 Email:

uksales@iskushealth.com

www.iskushealth.com